



Autoclave Monthly Maintenance Protocol

Updated/verified 3/9/2020

*No instruments should be sterilized during this time

1. Drain water through the hose, located on the left side of the door, into a bucket.
2. Pour 1 ounce of Speed Clean into spout and refill reservoir to about 1 inch above red line
3. Run 1 cycle on unwrapped/pouch cycle
4. Drain reservoir through hose into bucket
5. Refill reservoir with distilled water to about 1 inch above red line
6. Run 1 cycle on unwrapped/pouch cycle
7. Allow to cool down
8. Drain reservoir through hose into bucket
9. Allow to cool
10. Remove trays and rack, scrub with warm soapy water
11. Pull out small screens and rinse with distilled water, put screens back in
12. Remove door gasket and clean with warm soapy water
13. Log on clipboard beside autoclave