

Fluid Pump Monthly Maintenance
1/20/21 AM

For Heska Vet/IV 2.2 Infusion Pump

Use only Baxter brand IV lines

To check accuracy, Set pump to run 100ml of fluids over 1 hour.

Attach IV bag and line as you would for normal use

Open the end of the fluid line (What would normally attach to needle or IV catheter)

Place open end in a bowl large enough to hold 100ml of fluid

Allow pump to run

Check back in 1 hour and measure fluid in the bowl with a syringe to check accuracy.

Should be within 5% accuracy so if running 100ml over 1 hour you should have between 95 -105 ml

If the pump is not within 5% accuracy, call Heska for directions on how to send the pump in for repair